

In addition to helping clean ears, regular use of the Ear Irrigator can help prevent ear wax buildup by using product up to three times per week. More frequent use may be needed if treating heavier wax buildup.

Warning: If you experience dizziness or pain, discontinue use. Do not resume irrigation without consulting a doctor. Never irrigate ears if eardrum is known or suspected to be perforated, if tympanostomy tubes are present, or if any infection, drainage, bleeding, pain or irritation is present. Not for use in children under three (3) years of age.

Impacted ear wax should be removed by a health care professional before using the Ear Irrigator.

Store at room temperature (59-86°F; 15-30°C). Do not freeze.

Contents:
Deionized Water, Sodium Chloride,
Benzalkonium Chloride



Getting Started: Remove protective cap from bottle by turning counter-clockwise. Place Tri-Stream Tip on bottle and turn clockwise to tighten.

Cleaning the Ear: Always irrigate in a seated position, holding a small bowl or basin under the ear to catch exiting solution and debris. The head should be tilted slightly toward the ear to be cleaned. Gently insert the Tri-Stream Tip into the ear. Slowly squeeze the bottle to irrigate the ear. Repeat process until the ear is clean. Remove tip and thoroughly clean after each use. Replace protective cap and tighten onto bottle.

Cleaning and Storage: Clean tip thoroughly after each use with warm soapy water. Rinse well and let air dry. Store in a cool, dry place.

Health Enterprises, Inc. North Attleboro, MA 02760
1-800-633-4243 www.healthenterprises.com
Made in the USA