
ACU-LIFE®


ACU-STRAP™ MORNING & MOTION SICKNESS BAND

The Acu-Strap™ is intended for the relief of nausea. Nausea may be experienced due to Travel (Motion Sickness), Pregnancy (Morning Sickness), Anesthesia or Chemotherapy.

The Acu-Strap™ operates by exerting pressure on the Nei-Kuan (acupressure) point on each wrist. To find your Nei-Kuan point, place your middle three fingers on the inside of your wrist with the edge of the third finger just below the first wrist crease.

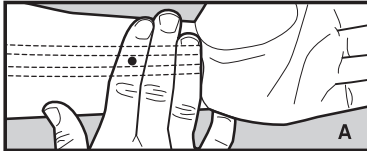The Nei-Kuan point is located underneath your first finger in between the two wrist (flexor) tendons (A).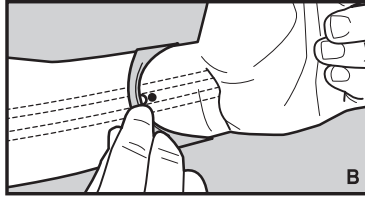

Position the button facing inward over the Nei-Kuan point (B). Repeat the process for the other wrist.

One band must be worn on each wrist to be effective. Effectiveness may vary with individual patient characteristics. May take up to five minutes to become effective, but usually lasts as long as you wear the bands. For better results put bands on before starting travel. Greater relief can be obtained by applying finger pressure directly to the button. The Acu-Strap™ should only be used on the Nei-Kuan point. Use at any other location will not relieve nausea.

The Acu-Strap™ may be washed with mild detergent in warm water up to five times. Replace when the bands no longer fit snug around the wrist.

Warnings:

- The Acu-Strap™ provides
 - symptomatic relief, it will not cure
 - or remove the cause of the nausea.
- Nausea and vomiting can be
 - associated with serious conditions.
 - If your symptoms persists, consult a
 - physician at once.
- If skin irritation occurs beneath the
 - band, or if pain or swelling occurs,
 - remove the bands immediately
 - and consult a physician if these
 - symptoms persist.
- Store out of reach of children. The
 - button is a choking hazard if
 - removed.

Individuals with known or suspected circulation problems should consult a physician prior to use.

 Health Enterprises, Inc.
90 George Leven Drive
North Attleboro, MA 02760
www.healthenterprises.com
info@healthenterprises.com
For more unique healthcare products,
visit us at www.healthenterprises.com

ACU-LIFE®

ACU-STRAP™ MORNING & MOTION SICKNESS BAND

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